## English

### Thermistor Airflow Sensor

## Intended Use

To detect respiratory airflow during sleep disorder investigations in a clinically controlled environment, such as a sleep laboratory, hospital, or other environment only when prescribed by a physician. These devices are for pediatric or adult patients.

## Caution

U.S. Federal law restricts this device to sale by or on the order of a physician.

## Warnings

This product is for diagnostic purposes only and is not to be used as an apnea monitor or in a life supporting or life sustaining situation.

Pro-Tech airflow sensors are sensitive medical devices and should be handled with care to avoid damage.

Pro-Tech thermistor airflow sensors are sold non-sterile and must be cleaned prior to each use. Refer to the Cleaning section of this User Guide for more information.

## WEEE-RoHS/Disposal

✂ If applicable, separate collection for electrical and electronic equipment per EC Directive 2002/96/EC. Otherwise, dispose of this device in accordance with local regulations.

Specifications	
Operating Temperature	5°C (40°F) - 30°C (86°F)
Storage Temperature	-20°C (-4°F) - 60°C (140°F)
Operating/Storage Humidity	15 - 95% Non-condensing

## Sensor Installation

- Place sensor on the patient as shown on Figure 1.
- Secure in place using the cinch tube and medical tape.
- Gently bend the sensor tips to position them to receive maximum patient airflow.
- Avoid having the sensor tips contact the patient’s skin. Skin contact will significantly attenuate the airflow signal.

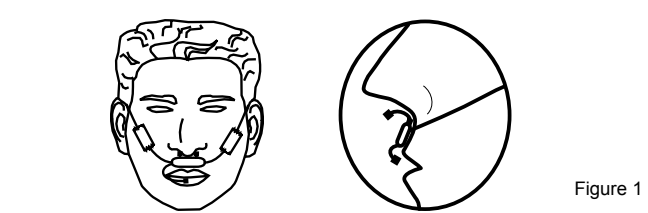
Figure 1

Plug the connector(s) into the appropriate jack(s) on the recording device. Refer to the recording device's instructions for use to determine channel settings.

## Cleaning

To clean before each use:

- Wash sensor body, sensor tips and wire set with soapy water, rinse thoroughly and towel dry. Avoid getting the connector(s) wet during this process.
- Allow the sensor to dry completely before reuse.

**Note:** Steam autoclaving is not recommended for Pro-Tech airflow sensors.

## Warranty

Pro-Tech Services, Inc. warrants this product to be free from defects in materials and workmanship for a period of fifty uses or six months from the date of sale by Pro-Tech Services, Inc., whichever comes first. If the product fails to perform in accordance with this warranty, the sole liability of Pro-Tech Services, Inc. is limited to repair or at its option replacement of the product with no charge for parts or labor. Under no circumstances shall Pro-Tech Services, Inc. be liable for any indirect, special or consequential loss or damage, including economic loss, loss of business or overhead, or any punitive damages, arising from the use of, or the inability to use its product. This warranty is rendered void and Pro-Tech Services, Inc. shall not be liable for conditions resulting from damage, marginal performance or malfunctions caused by misuse, abuse, neglect, improper line voltage, power fluctuations, or any adverse environmental conditions, tampering, unauthorized modifications, adjustments or repairs to the product or its accessories, or any other defect not related to materials or workmanship. This warranty is in lieu of all other warranties, express or implied, and is extended only to the original purchaser. Features and specifications are subject to change without notice.

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