**zRIP™ Effort Sensor  
User Guide**

## **zRIP™ Effort Sensor User Guide**

Pro-Tech, a Respironics company

Tel: 800-345-6443, US & Canada

724-387-4000, International

Fax: 800-886-0245, US & Canada

724-387-5012, International

Email:

[clinical@respironics.com](mailto:clinical@respironics.com), US

[Sleepdiagsupintl@respironics.com](mailto:Sleepdiagsupintl@respironics.com), International

Manufactured by:

Pro-Tech Services, Inc

4338 Harbour Pointe Blvd SW

Mukilteo, WA 98275 USA

**EC REP** Advena Ltd. Hereford HR4 9DQ UK

POM9486, Rev. NEW

### **Intended Use**

zRIP Respiratory Effort Sensors are intended for use during sleep disorder studies to detect respiratory effort for recording onto a physiologic recorder. zRIP effort sensors may be used as stand alone sensors with compatible RIP systems or with the Pro-Tech zRIP module.

### **Warnings**

The zRIP System is not to be used in the presence of flammable anesthetic mixture with air or with oxygen or nitrous oxide.

To minimize toxic hazards to the environment, dispose of the batteries according to the manufacturer's instructions and in accordance with local regulations. Pro-Tech Services, Inc. is not responsible for the disposal of the batteries.

Do not operate the zRIP System with the battery cover open.

zRIP modules and sensor belts must be connected to an electrically isolated input.

Patient injury could occur if not connected properly.

### **Cautions**

This product is for diagnostic purposes only and is not to be used as an apnea monitor or in a life supporting or life sustaining situation. Federal law restricts this device to sale by or on the order of a Physician.

### **Specifications**

Adult, Large, size	45" - 123" (114cm - 312cm)
Adult, size	24" - 75" (61cm - 190cm)
Pediatric, size	16" - 42" (41cm - 107cm)
Operating Temperature	5°C (40°F) - 40°C (104°F)
Storage Temperature	-20°C (-4°F) - 60°C (140°F)
Operating/Storage Humidity	15 - 95% Non-condensing

### **System Components**

Typical zRIP System Components (sold separately or in kits):

- zRIP Effort Sensor, Adult, Large, or
- zRIP Effort Sensor, Adult, or
- zRIP Effort Sensor, Pediatric
- zRIP Wireset
- zRIP Module

Two zRIP effort sensors are typically used with each patient, one for the thorax and one for the abdomen. zRIP wiresets are used to connect the zRIP effort sensors to the zRIP module which is then connected to the physiological recorder. However, some physiological recorder systems may accept the zRIP effort sensors without the use of the zRIP module.

### **zRIP Effort Sensors**

The zRIP effort sensors are placed comfortably, but securely around the abdomen and thorax and when stressed by the expansion and contraction of respiration generate a voltage signal which is conditioned to provide a respiratory effort tracing onto a physiologic recorder.

### **Sensor Installation – Belt Placement**

To apply, adjust the belt so it is approximately the size of the patient:


#### **Adjustments**

##### **To lengthen the belt:**

Hold the male buckle, pull the inner layer of the loop down and away from the buckle, then grasp the belt past the slide lock and pull them apart until the loop is flat.

### To shorten the belt:


Hold the slide lock and pull the inner layer of the loop toward the male buckle, then grasping the slide lock and the male buckle, pull them apart until the loop is flat. Snap buckles together in the front. Adjust the belt so that it is secure, but not overly tight: To loosen the belt, grasp the slide lock and lift slightly away from the patient while pulling towards the loop; To tighten the belt, grasp the slide lock and pull away from the loop while repeatedly grasping and pulling the outer layer of the loop toward the slide lock.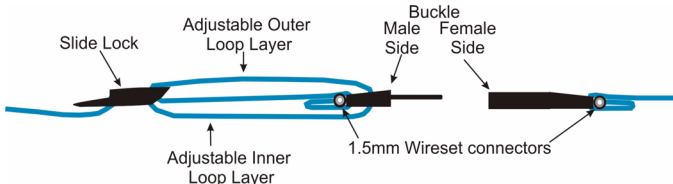

**Buckle, Slide Lock, and Adjustable Loop**

**Figure 1**

1. Adjust as above.
2. Plug the 1.5 mm end of the wireset into the belt near the buckles; plug the other end, with a 249 connector into the designated inputs on the zRIP module. The blue plug is signal and the white is reference/common.
3. Plug the output channels into inputs of your polygraph.

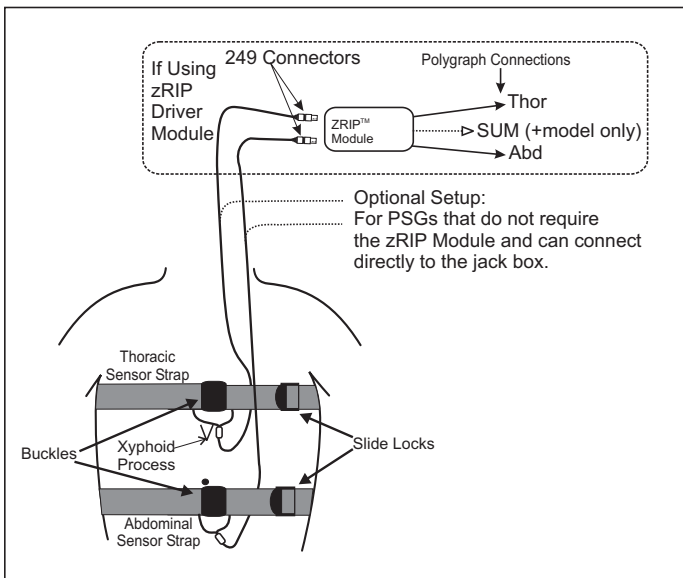**Figure 2**

### Instrument Settings

**Sensitivity** – Approximately 50 $\mu$ V/mm. Adjustment of the sensitivity up or down is typically required. Response is dependent upon such variables as sensor application and patient effort.

**Low Frequency Filter / Time Constant** - 0.16 Hz (or 1 second or longer). Shorter time constants or higher low frequency filter settings will significantly attenuate waveforms.

**High Frequency Filter** – 35 Hz.

### **Cleaning**

Wash belts using warm water and household laundry detergent, air dry. Care should be taken to ensure drying of the safety plugs after cleaning. Do not clean belts with alcohol or alcohol based products.

The wire set may be wiped down with soapy water or a bleach or alcohol towlette. Do not immerse in liquids.

### **Warranty**

Pro-Tech Services, Inc. warrants zRIP effort sensors to be free of defects in materials and workmanship for a period of one year, and zRIP wiresets for a period of 6 months, from the date of purchase. Should any product prove defective in workmanship, materials or performance during the warranty, the sole liability of Pro-Tech Services, Inc. is limited to repair or, at its option, replacement of the product with no charge for parts or labor. Under no circumstances shall Pro-Tech Services, Inc. be liable for any loss or damage, direct, consequential, or incidental, including property damage or personal injury arising from the use of, or the inability to use this product. This warranty is rendered void and Pro-Tech Services, Inc. cannot be held liable for conditions resulting from: damage, marginal performance or malfunctions caused by: misuse, abuse, neglect, improper line voltage, power fluctuations, or any adverse environmental conditions, tampering, unauthorized modifications, adjustments or repairs to the product or its accessories. This warranty is in lieu of all other warranties, expressed or implied, and is extended only to the original purchaser. Features and specifications are subject to change without notice.

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